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IDEALNY NA PREZENT !!!

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1. Alarm is initially turned off by default. Press and hold the power button to switch between 'On' and 'Off'. There will be a "Dial" alarm prompt after ALM turned on.

2. SpO2 indicates the alarm value setting of the pulse oximeter. Long press the power button to increase or decrease the alarm value. If the set value is higher or lower than the set value, there will be an alarm prompt.

3. PR indicates the alarm value setting of the pulse oximeter. Long press the power button to increase or decrease the alarm value. If the set value is higher or lower than the set value, there will be an alarm prompt.

4.3 Parameter setting: Press the power button (in 0.5s), the oximeter will enter into the parameter setting.

4.4 Possible problems and effective solutions

Problem	Possible reason	Solution
SpO2 or PR does not show	1. Finger is not plugged correctly 2. Finger is too thick or too thin 3. Oximeter value is too low to be measured	1. Replug the finger 2. Try another finger, if you can't find any finger to fit, please go to hospital for diagnosis
SpO2 or PR is shown unstably	1. The finger might not be plugged deep enough 2. Finger is trembling or the patient is in movement status	1. Replug the finger 2. Please remain at rest
To Oximeter does not be turned on	1. Inadequate power or power off 2. Batteries might be installed incorrectly 3. The Oximeter might be damaged	1. Please replace the batteries 2. Please correct the batteries 3. Please contact with local customer service center
Indication lamp is suddenly off	1. The product automatically shuts off when no signal is detected in 5 seconds 2. Inadequate power	1. Manual 2. Replace the batteries

Features

- Two color OLED display, more display modes
- Low voltage indicator
- Automatically power off in 5 seconds when there is no signal

4.1 Classification

Equipment

Anti-electric Shock Type: Internally powered equipment

Anti-electric Shock Degree: Type BF

4.2 Maintenance and Preservation

Replace the batteries timely when low voltage lamp is on.

Clean the surface of fingertip oximeter before it is used to diagnose patients.

Remove the batteries inside if you will not use it.

4.3 Product declaration

Reference and manufacturer's declaration electromagnetic radiation for other EQUIPMENTS and SYSTEMS

Equipment	Declaration
SpO2	Not applicable
PR	Not applicable
SpO2 and PR	Not applicable

4.4 Possible problems and effective solutions

Problem	Possible reason	Solution
SpO2 or PR does not show	1. Finger is not plugged correctly 2. Finger is too thick or too thin 3. Oximeter value is too low to be measured	1. Replug the finger 2. Try another finger, if you can't find any finger to fit, please go to hospital for diagnosis
SpO2 or PR is shown unstably	1. The finger might not be plugged deep enough 2. Finger is trembling or the patient is in movement status	1. Replug the finger 2. Please remain at rest
To Oximeter does not be turned on	1. Inadequate power or power off 2. Batteries might be installed incorrectly 3. The Oximeter might be damaged	1. Please replace the batteries 2. Please correct the batteries 3. Please contact with local customer service center
Indication lamp is suddenly off	1. The product automatically shuts off when no signal is detected in 5 seconds 2. Inadequate power	1. Manual 2. Replace the batteries